

Assembly

Richard Wolf Flexible Endoscopes

1



- Inspect scope, accessories.
- Gather scope and accessories.
- Inspect cleanliness.
- Functionality
 - Light capabilities
 - Activate levers, distal tip

2




- Dry channels.
- Filtered air
- 70% alcohol

3

Choices to make:

Disinfection?

- Gas cap off!
- Manual
- Machine
- ↓
- Follow hospital protocol

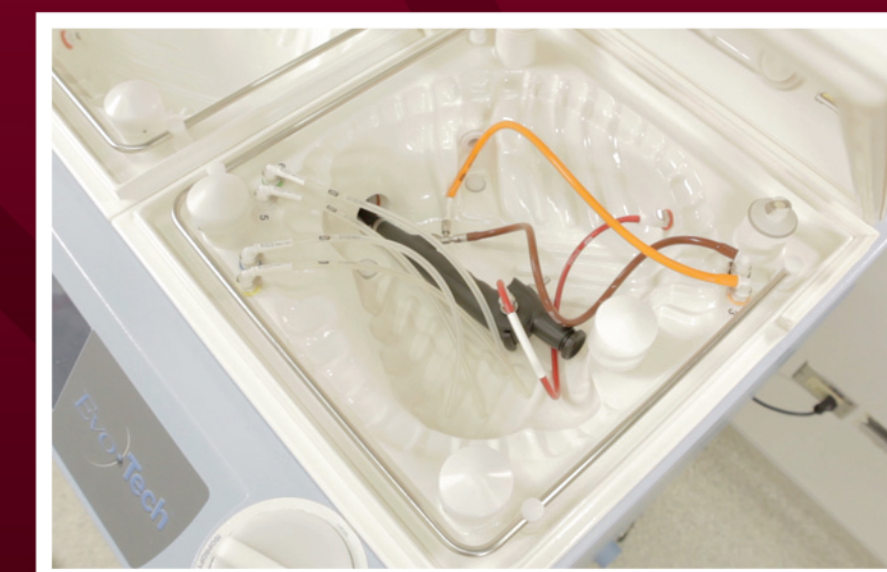

Terminal Sterilization?

- Gas cap on!
- Wrapped or containerized tray
- Terminally sterilize
- Store in secure sterile storage

