

# Disinfection

## Richard Wolf Flexible Endoscopes

### Manual Disinfection

1



- Scope is completely dry!
- Immerse completely:
  - Scope channels, accessories.
- Cover container, prevent chemical fumes in area.
- Use correct exposure time, temperature.
- Rinse scope and channels.
- Dry channels with syringe or compressed air (correct pressure, filtered) and 70% alcohol.
- Choices to make:
  - Patient use?
  - Clean storage?
- Follow hospital protocol for each choice

### Machine Disinfection

2



- First, perform leakage test, then manually clean.
- Connect scope correctly to machine.
- Follow machine IFU for operation.

### Clean Storage

3



- Inspect scope before storing.
- Scope is dry.
- Scope is tagged!
- Hang vertically, minimal contact, prevent cross contamination and damage!