

From the operating room
to the office.




**Diagnostic and Operative
Hysteroscopic Instrumentation**

Bringing care and quality to women's health.

From the OR to the office, our high-precision technologies help you achieve the best clinical outcomes for your patients. Richard Wolf's gynecologic endoscopy portfolio features trusted solutions, such as the 3.8 mm Compact Hysteroscope and the ENDOCAM® Performance HD Imaging System, streamlining the transition to office-based procedures.

Our advanced instruments are setting new benchmarks in patient safety and ease-of-use.

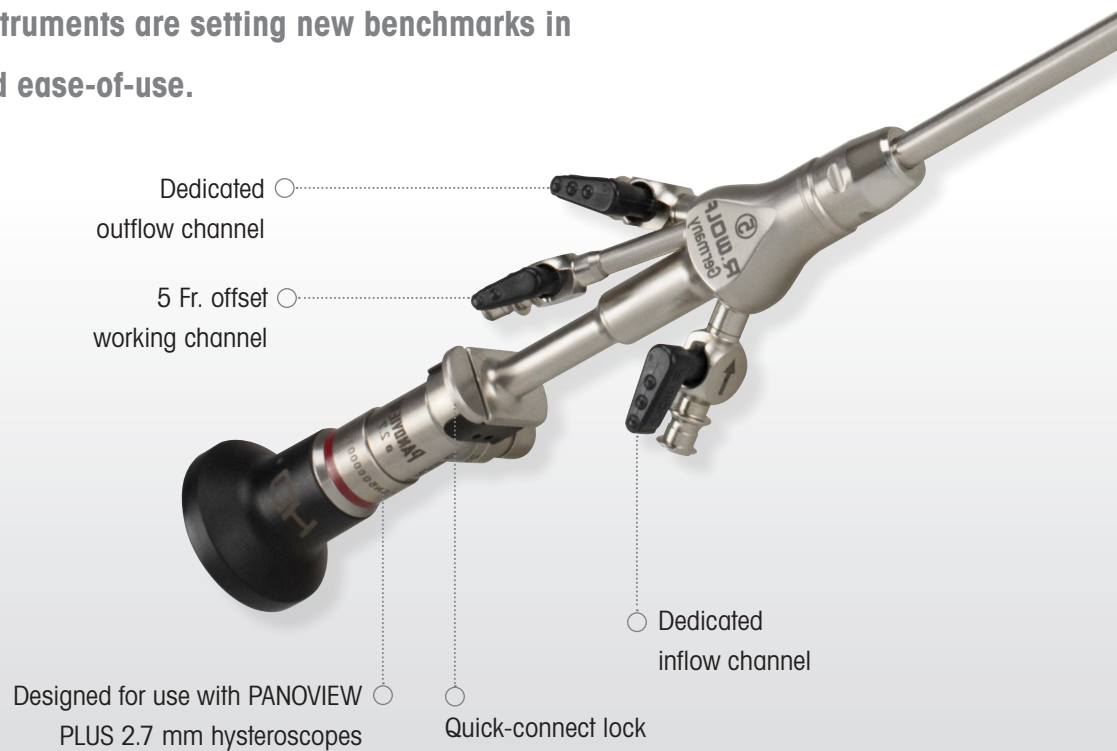

Optiflow

Continuous-Flow Sheath

Increased flow dynamics, even with an instrument inserted.

5.5 mm outer diameter

Oblique, atraumatic tip

Developed with increased flow dynamics compared to the traditional dual sheath system, the Optiflow™ Continuous-Flow Sheath features a single piece design, with a quick-connect lock for simple assembly of the hysteroscope. The 45° angle inflow and outflow ports are ergonomically designed for placement of tubing and access to the entire working length.

The built-in 5 Fr. offset working channel accommodates a broad range of instruments and the atraumatic, oblique tip provides easier access, even with a stenotic cervix.



- Oblique tip allows for atraumatic cervical access, even in stenotic patients
- Continuous flow sheath in a single piece design
- Ideal for operating room or office procedures
- Easy set up

Compact Hysteroscopes

3.8 mm and 5 mm

True, continuous flow for brilliant images.


Designed with a rod lens optics system, our Compact Hysteroscopes deliver brilliant, detailed images. These all-in-one, single piece hysteroscopes feature a 5 Fr. built-in working channel for expanded treatment options as well as separate inflow and outflow channels for optimal visibility.



Ideal for the office.

- With its slim diameter, the 3.8 mm Compact Hysteroscope promotes increased patient comfort and is therefore ideal for use in the office space.


- Ideal for the highly atraumatic treatment of fertility and sterility patients

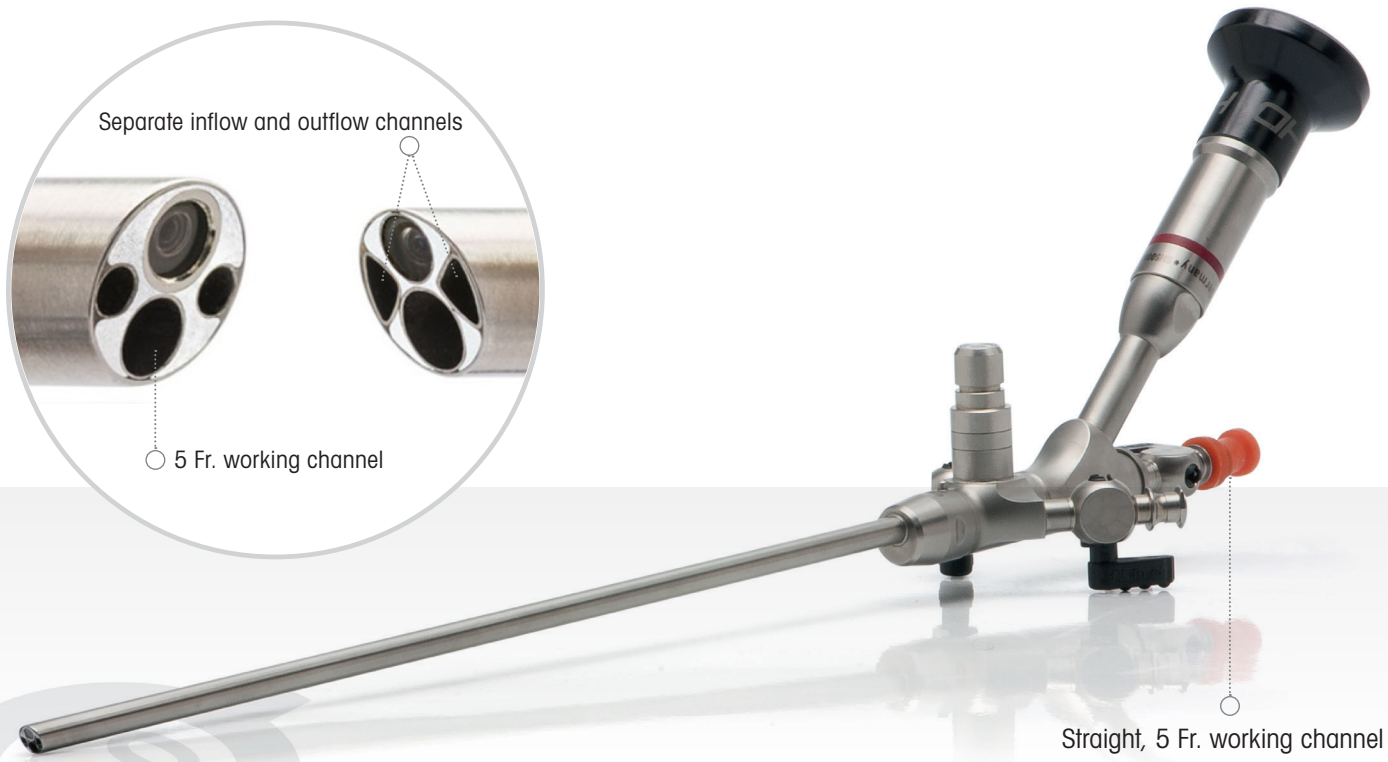

Straight, 5 Fr. working channel accommodates a broad range of instruments

Use with the Richard Wolf Ergonomic Handle.

- Ergonomic and easy to handle
- Fits the 3.8 mm and 5 mm Compact Hysteroscopes

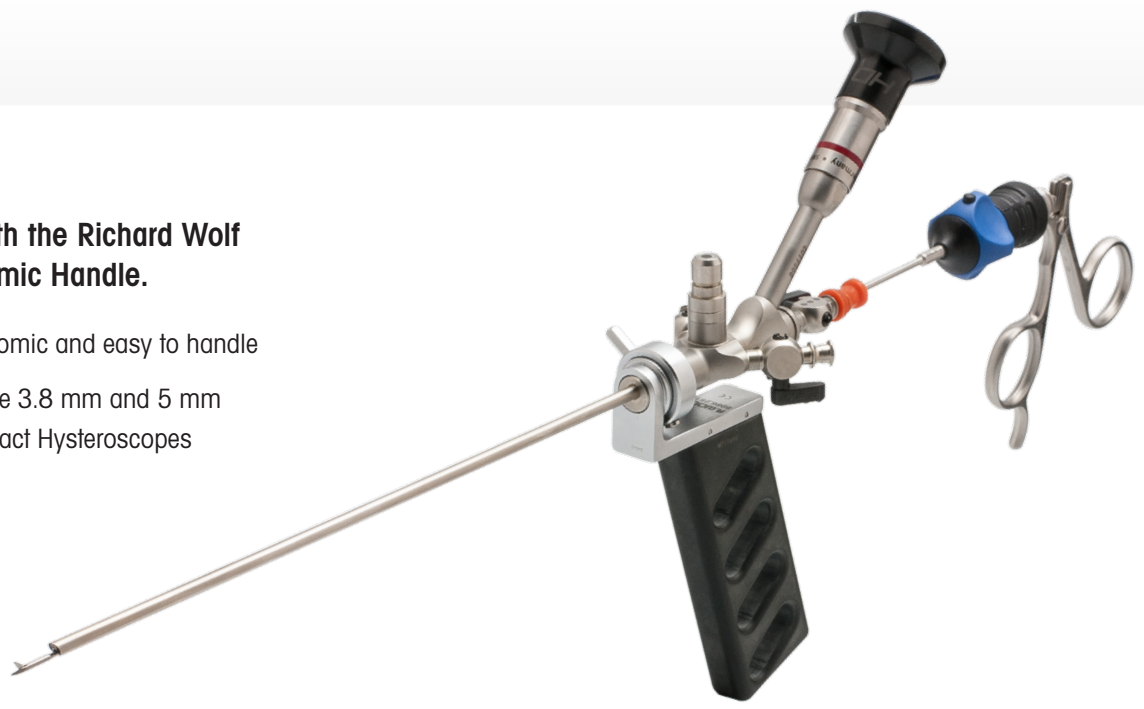

HySafe Micro-Modular Hand Instrument System


360° rotatable, flushable, and easy to assemble.



The 2-piece modular design makes the 5 Fr. HySafe hand instruments an economical choice as it facilitates reprocessing and replacement of the individual components. The instrument clicks together easily in any position and disassembles quickly to flush forceps and scissors. The jaws rotate 360° and location markers assist in jaw positioning. Additionally, the overload protection feature built into the handle extends the service life of the instrument.

5 Fr. Single Piece Semi-Rigid Forceps

A wide variety of jaw patterns for various applications.

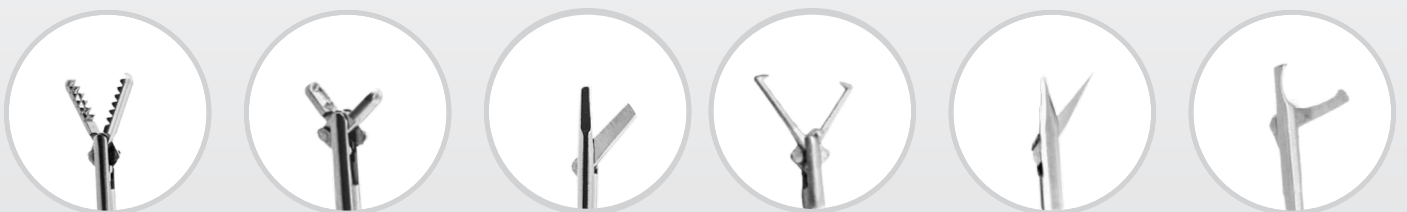

ENDOCAM® Performance HD

Maximize performance, minimize costs.

Simplicity in operation

The intuitive functionality makes the ENDOCAM Performance HD plug-and-play system particularly easy to use.

The white balance feature can be controlled from the CCU or camera head giving flexibility to the surgical team.



Brilliance in view

HD technology delivers pin-sharp images and high light sensitivity ensures clarity in each detail.



USB storage

ENDOCAM Performance HD saves images and videos to a USB stick at the touch of a camera head button.

ENDOCAM Performance HD offers a true HD image at a cost-effective price.



Office Hysteroscopy Packs

Disposable packs for in-office patient procedures.

Standard Office Hysteroscopy Pack

(Sterile and disposable — 10 sets per box)

- Under buttocks drape
- Half drape
- Camera drape
- Table cover
- Alcohol swab
- Hand towels
- Leggings
- Prep kit:
 - 2 large PVP sponge slickers
 - Cysto tubing set
- Extension IV tubing set



Compact Office Hysteroscopy Pack

(Sterile and disposable — 10 sets per box)

- Under buttocks drape
- Half drape
- Camera drape
- Table cover
- Cysto tubing set
- Extension IV tubing set

Ordering Information

Product Description	Part Number
Sheaths & Hysteroscopes	
Optiflow 5.5 mm Hysteroscope Sheath Bundle	89743516
PANOVIEW PLUS Telescope, 12°, Ø 2.7 mm, SL 310 mm	8974.412
PANOVIEW PLUS Telescope, 30°, Ø 2.7 mm, SL 310 mm	8974.402
3.8 mm Compact Hysteroscope	8986402
5 mm Compact Hysteroscope with 5 Fr. Channel Bundle	89864016
Hand Instruments	
HySafe 5 Fr. Grasping Forceps Bundle (Alligator)	8642.6502
HySafe 5 Fr. Grasping Forceps Bundle (Spoon Biopsy)	8642.6002
HySafe 5 Fr. Scissors Bundle	8642.6812
HySafe 5 Fr. Biopsy Forceps Bundle (Punch Biopsy)	8642.6312
5 Fr. Grasper, Double Action, Single Piece (Alligator)	86422436
5 Fr. Cup Biopsy, Double Action, Single Piece	86422430
5 Fr. Pointed Scissors, Single Action, Single Piece	86422335
5 Fr. Scissors, Single Action, Single Piece (Blunt)	86422434
5 Fr. Hook Scissors, Single Action, Single Piece	86422416
5 Fr. Tenaculum, Double Action, Single Piece	86422341
Imaging System	
ENDOCAM Performance HD Bundle	551465515
ENDOLIGHT Light Source LED 1.1, 76W Bundle	51600115
RIWO Zoom Lens, f = 13-29 mm	5261504
RIWO Lens, f = 24 mm	85261244
Fiber Light Cable Bundle, 2.5 mm	806625301
RIWomobil mini Video Cart	RWF3000
21.5" HD LCD Monitor Bundle	MDRAP215
Printer UP-25MD	5365.250
Recommended Accessories	
Handle for Hysteroscope	8986.251
Hysteroscopy Pack (Standard)	4974.0010
Hysteroscopy Pack (Compact)	4975.0010
2.7 mm Hysteroscope Tray, Plastic (2.7 mm Telescope and Accessories)	8570.027
Compact Hysteroscope Tray, Plastic (3.8 or 5 mm Compact Hysteroscope and Accessories)	8570.040

To arrange a trial, contact your local Richard Wolf representative or call Customer Service at (800) 323-9653.