

Quick Guide

for manual Reprocessing

Flexible Endoscopes from Richard Wolf



The Quick Guide is an overview. It does not replace the specific instructions for use of the respective flexible endoscope. The respective instructions for use must be observed. The Quick Guide can only be used as a supplement to the instructions for use.

Preparation / Leakage test	1	2	3	4
	5	6	7	8
	9	10	11	12
	13	14	15	16
Manual Cleaning	17	18	19	
	20			
Rinsing Drying	21		22	
	23			
Disinfection Rinsing	24		25	
	26			

Drying Maintenance	19	20	21	22	...we recommend:
	23				
Sterilization H ₂ O ₂ , LTSF, EO	24		25		<ul style="list-style-type: none"> • RIWO-Box Disinfection basin 600x400mm, REF 68603 • RIWO-Box Disinfection basin 880x200mm, REF 68602 • Water jet cleaning gun incl. mounting, REF 6199.00 • Leakage tester for flexible endoscopes, REF 163.903 • Pressure equalization valve, REF 163.904 • Cleaning brushes see separate table
	26				
Packaging Storage	27		28		
	29				

Recommended Cleaning Brushes	Diameter in mm	Color	Brush length in mm	Total length in mm	Application		Type	
					in working & laser channel	in the flexible endoscopes		
Cleaning brush for single use (pack = 10 pcs.)	2	Yellow	15	1200	∅ 1,0 - 1,5 mm	7223.xxx 7325.xxx 7326.xxx	7330.xxx 7331.xxx	7990001
	3	Green	20	1200	∅ 2,0 - 2,5 mm	7250.011 7265.xxx 7268.xxx 7269.xxx 7270.xxx	7305.xxx 7308.xxx 7309.xxx 7310.xxx	7990002
	0,85	Blue	10	1200	∅ 0,8 mm	7356xxx		7990003
	5	White	20	1200	∅ 2,5 - 3,5 mm	7235xxx 7236xxx 7260xxx 7315xxx		7990005
Cleaning brush straight, for single use	-	Black	25	-	-			8691