

Competency Guide: Disassembly and Assembly of **ERAGON** modular Instruments in the Sterile Processing Department

This form is used to measure the user's competency to correctly assemble and disassemble Richard Wolf ERAGONmodular instruments in the sterile processing department. The activities needed to correctly assemble the instrument insert, sheath tube and handle are based on Richard Wolf IFU GA-S027, Section 7.




In	structions		Practice to Policy	Needs Assistance	Reviewer's Initials/Date
			1	1	
Disas Step	ssembly of ERAGONmodular	Instruments			
1a	Grasp fully assembled ERAGONmodular instrument with two hands.				
1b	Disengage ratchet (if present) and open handle completely. Ensure hands do not restrict handle from opening.				
1c	When ratchet is fully disengaged (if present), depress black button on rotation knob.				
1d	Separate sheath tube from handle.				
1e	Place handle aside.				
Step	2				
2a	Gently twist jaw counterclockwise to uncouple from sheath tube.	4			
2b	Grasping the closed jaw tip, slide insert out of sheath tube.	Hunts			
2c	Place both sheath tube and jaw insert aside.				

In	structions		Practice to Policy	Needs Assistance	Reviewer's Initials/Date
			/	/	
Asse Step	mbly of ERAGONmodular Inst 1	ruments			
1a	Guide instrument insert into sheath tube.				
1b	Gently twist jaw clockwise to lock insert into sheath tube. Pull on jaw tip to confirm insert is locked into sheath tube.	C			
Step	2				
2α	Select handle with appropriate functionality for instrument (i.e. ratcheted/non-ratcheted, HF post/without-HF post).	200			
2b	Disengage handle ratchet (if present). Open handle fully.				
2c	Grasp handle behind gray rotation knob; ensure hands do not restrict handle from opening fully. Insert sheath into open handle. Sheath and handle should click into place. Only slight effort will be required if assembled correctly.				
2d	Confirm successful assembly by opening and closing handle. Watch for corresponding jaw motion.				

Staff Person's Name, Title	Signature
Primary Educator or Preceptor's Name, Title	Signature
Secondary Educator or Preceptor's Name, Title	Signature



spirit of excellence

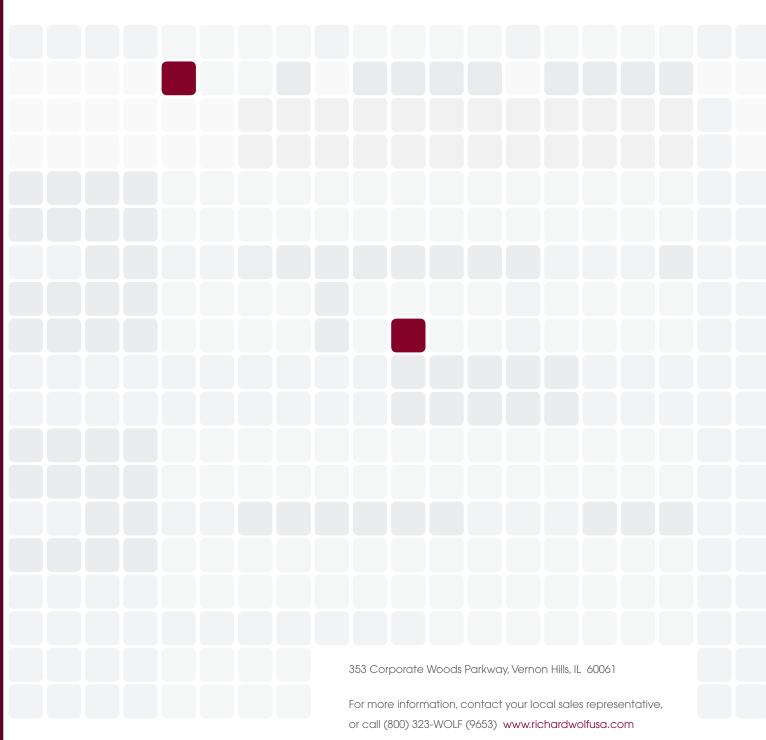



@2016 Richard Wolf Medical Instruments . All Rights Reserved.