



# ERAGON<sup>modular</sup> 3.5 mm, 5 mm, 10 mm LAPAROSCOPIC INSTRUMENTATION

## Disassembly


### Disassembling the sheath tube and handle

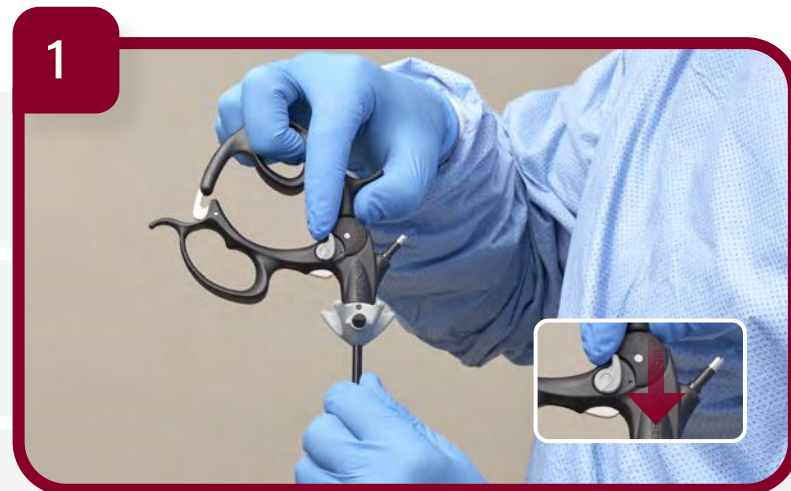

- ▣ Deactivate the handle lock (if present)

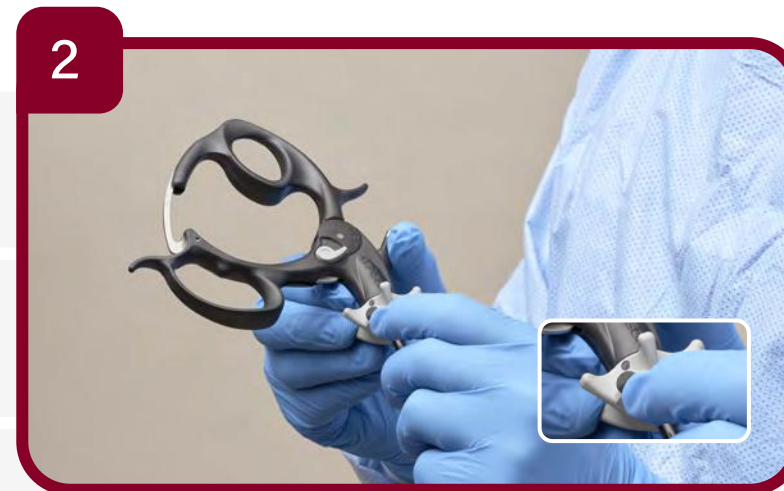

- ▣ Press the black button on the gray rotation knob
- ▣ Do not hold the movable grip during disassembly

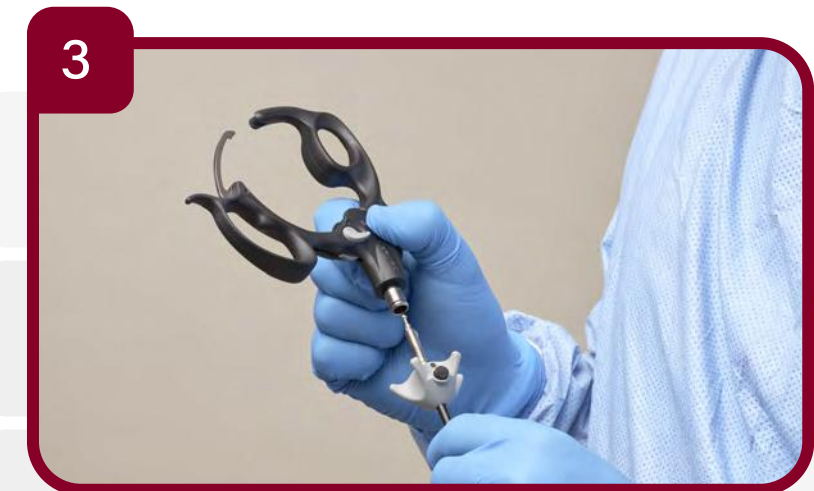


- ▣ Separate the sheath tube from the handle

### Disassembling jaw insert and sheath tube



- ▣ Turn the jaw insert in direction of arrow (counterclockwise) to uncouple from sheath tube

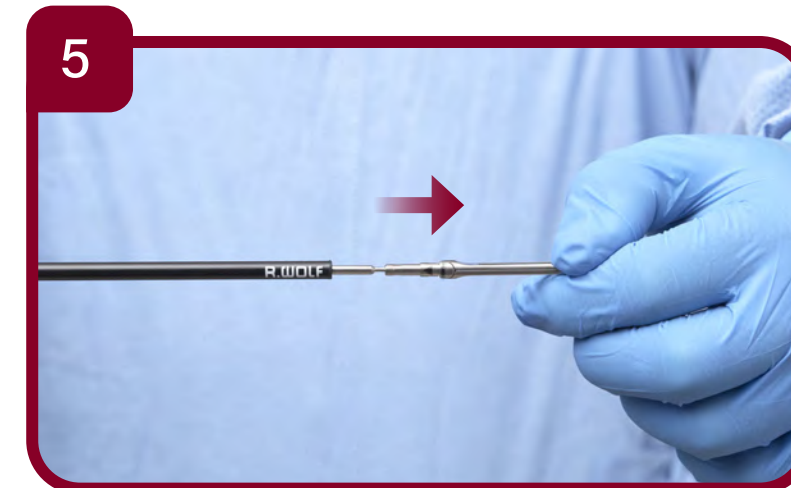

- ▣ Pull insert out of the sheath tube