


Reprocessing of Rigid Endoscopes

Decontamination Area



- 1 Rinse, disassemble scope from assemblies, scope attachments
- Soak in enzymatic detergent solution
- Flush channels with enzymatic solution before brushing scope accessories

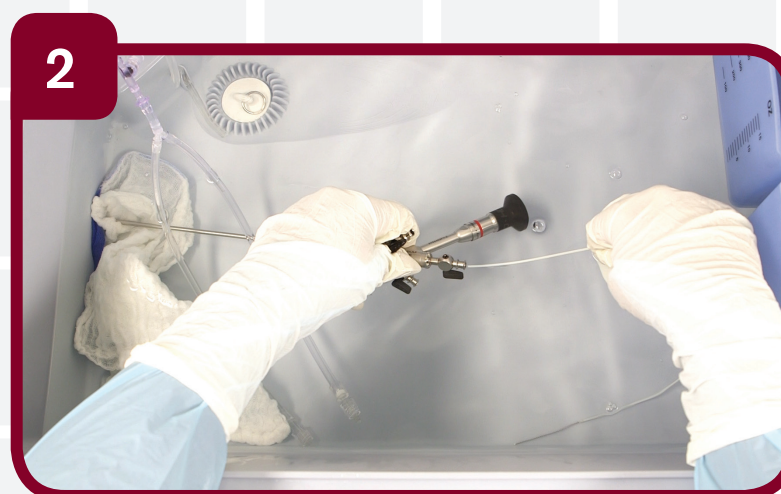

- 2 Brush channels and accessories with the correct size brush
- Flush after brushing with enzymatic solution



- 3 Rinse channels using flush device, or with a 20 cc syringe
- Rinse scope channels, surface, and accessories with processed water
- Dry channels with filtered compressed air. May also use 5 cc of 70% alcohol to facilitate drying of channels.