

Reprocessing of Rigid Endoscopes

# Office Reprocessing



- ▣ Keep scope separated from instruments
- ▣ Soak in enzymatic detergent solution before cleaning



- ▣ Flush/brush/flush channels with enzymatic detergent solution and nylon brush
- ▣ Wash attachments and assemblies with enzymatic detergent solution, brush and lint-free cloth
- ▣ Rinse all detergent solution from channels and surfaces



- ▣ Dry channels and surface
- ▣ Inspect optics—view object at short focal point
- ▣ Inspect light output—view scope light post with scope pointed to ceiling
- ▣ Inspect light cable
- ▣ Attach scope protector
- ▣ Sterilize; according to IFU. Alternate reprocessing method for cleaning at [www.richardwolfusa.com](http://www.richardwolfusa.com)